

HRM Copper Masks



Caring for your mask

Your mask and liner (if removable) are both easy to care for and fully washable!

Always wash your mask with gentle detergent, and with water only as hot as your hands are able to tolerate. The best option for drying is air dry or out on the line. You can machine wash, preferably on a gentle / delicate cycle and tumble dry on low. A third option is to leave it on the dash to be sanitized by the sun! This is the least optimal option for elastic longevity.

1

HAND WASH

The best option is to hand wash your mask and filter with hot water and line / air dry

2


GENTLE WASH

Add to your delicate / gentle wash items on warm, tumble dry on low heat

3

SUN DISINFECT

Leave your mask in the sun on your dash between uses for natural disinfecting!

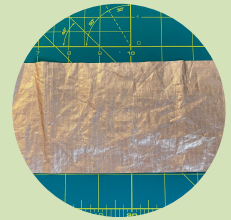

Copper Inserts:

Make sure if your mask comes with a copper insert, that the fabric side is the side facing the opening in the back of your mask.

Never have copper fabric in contact with your skin.

One-piece copper masks:

Your copper is sewn into your mask so you can just wear and go!